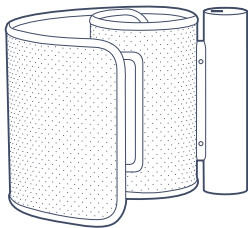
# Nokia BPM

## QUICK INSTALLATION GUIDE

---

SNABBINSTALLATIONSGUIDE | KVIK INSTALLATIONSVEJLEDNING | VEILEDNING FOR HURTIG INSTALLASJON | PIKA-ASENNUSOPAS

EN | SV | DA | NO | FI

# THANK YOU FOR CHOOSING NOKIA BPM

---

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This guide is available in additional languages at: [www.nokia.com/health/guides](http://www.nokia.com/health/guides)

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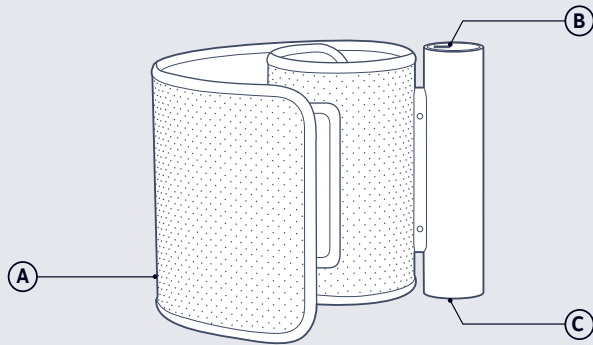
---

1

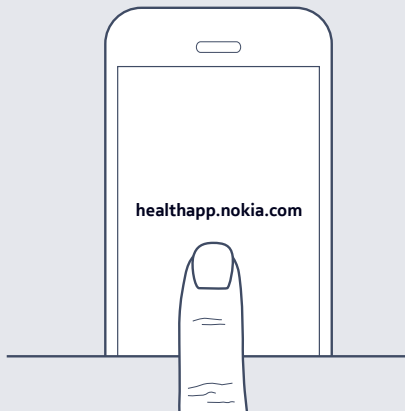
**Visit [healthapp.nokia.com](https://healthapp.nokia.com) from your smartphone or tablet, and follow the instructions to install the Health Mate™ app.**

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2

**In the app, follow the instructions to install Nokia BPM.**

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**NO** Sett deg ned i en komfortabel stilling.

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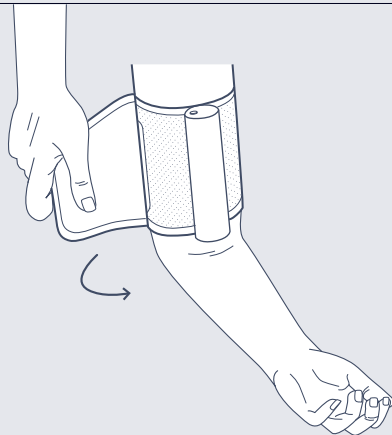
2

**Wrap the cuff around your arm. The metal tube should be positioned against your inner arm.**

- SV** Linda manschetten runt armen. Metallslangen ska vara positionerad mot armens insida.
- DA** Sæt manchetten rundt om armen. Metallrøret bør placeres imod din indre arm.
- NO** Plasser mansjetten rundt armen din. Metallrøret skal plasseres mot den indre delen av armen.
- FI** Kierrä mansetti käsivartesi ympärille. Metalliputki tulee asettaa vasten käsivartesi sisäpuolta.

# How to take a blood pressure measurement

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---

3

**Place your arm on a table and level with your heart.**

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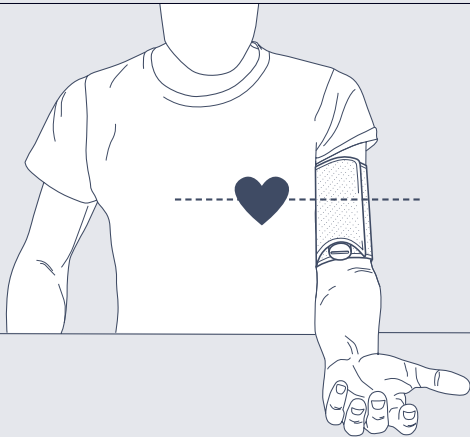
4

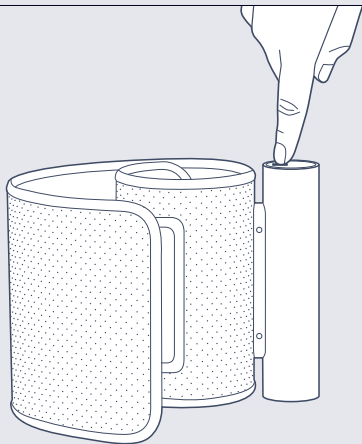
**Turn on your blood pressure monitor.  
The app starts automatically.**

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
## 5

**From the app, press the start button launch the mesurement.**

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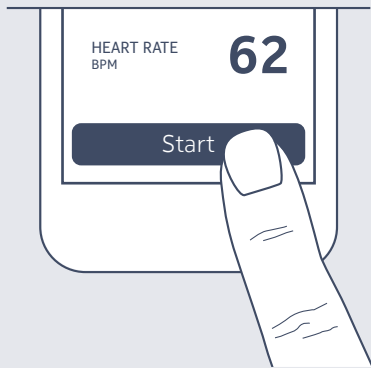12

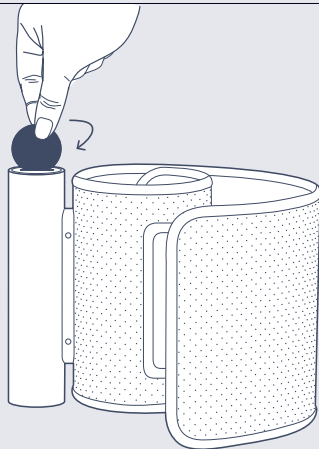
**Use a coin to remove the battery cover.**

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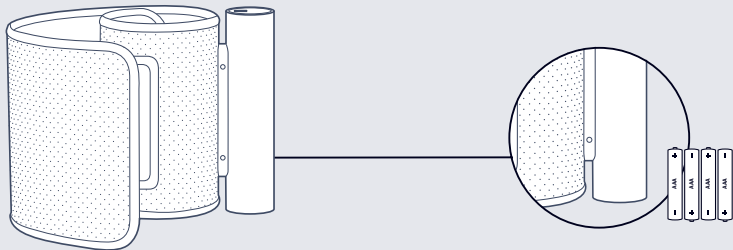
DA Blodtryksmonitor

NO Blodtrykksmåler

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## AAA alkaline cells (already inserted, X4)

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### EN This device complies with Part 15 of the FCC Rules and with Industry Canada licence-exempt RSS standard(s).

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications made to this equipment not expressly approved by Nokia may void the FCC authorization to operate this equipment.

#### Radiofrequency radiation exposure information :

The radiated output power of the device is far below the FCC radio frequency exposure limits. Nevertheless, the device shall be used in such a manner that the potential for human contact during normal operation is minimized.

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Frequency band :  
2402 - 2480 MHz  
Maximum output power :  
4.45 dBm

Withings (Part of Nokia) hereby declares that the device Nokia BPM is in conformity with the essential requirements and other relevant requirements of Directive 2014/53/EU.

The declaration of conformity can be found at: [nokia.com/health/compliance](http://nokia.com/health/compliance).

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➔ [nokia.com/health/support/bpm](https://nokia.com/health/support/bpm)

QIG\_WPM02\_020

**NOKIA**

Nokia BPM | Wireless Blood Pressure Monitor | v1.0

© 2017 Nokia Technologies Ltd. All rights reserved.