TENS/EMS Muscle Stimulator

SE | TENS/EMS-massageapparat
NO | TENS/EMS-massasjeapparat
FI | TENS/EMS-laite

Art.no. Model
34-1548 LT3011A

Ver. 20260302

English

Svenska

Norsk

Suomi

TENS/EMS Muscle Stimulator

Art.no 34-1548 Model LT3011A

Please read the entire instruction manual before use and save it for future reference. We reserve the right for any errors in text or images and for making any necessary technical changes to this document. If you should have any questions concerning technical problems please contact our Customer Services.

English

Table of Contents

1. Important safety precautions and warnings	5
2. The package contains	12
3. Product description	12
4. Display	14
5. Functions	15
5.1 <i>Treatments</i>	15
5.2 <i>Treatment areas</i>	15
5.3 <i>Treatment programmes</i>	15
6. Getting started	15
6.1 <i>Insert the batteries</i>	15
6.2 <i>Connect the cables to the electrode pads</i>	16
6.3 <i>Connect the cables to the TENS/EMS machine</i>	16
6.4 <i>Place the electrode pads on the skin</i>	16
6.5 <i>Switch on the TENS/EMS machine</i>	17
6.6 <i>Select the user</i>	17
6.7 <i>Choose a treatment and treatment area</i>	17
6.8 <i>Select the treatment programme</i>	17
6.9 <i>Start the treatment and adjust the intensity</i>	18
6.10 <i>Pause the treatment</i>	18
6.11 <i>Switch off the machine</i>	18
6.12 <i>Quick stop</i>	18
6.13 <i>Keypad lock</i>	18

6.14 Check what is stored in memory	19
6.15 Battery warning	19
6.16 Individual settings U1	19
7. Care and Maintenance	20
8. Troubleshooting	20
9. Responsible disposal	21
10. Specifications	22
11. Placement of the electrode pads	23
 Appendix/Bilaga/Liite 1 TENS	 99
Appendix/Bilaga/Liite 2 MASSAGE	100
Appendix/Bilaga/Liite 3 EMS	102
Appendix/Bilaga/Liite 4 EMC	103
Table 1	103
Table 2	103
Table 3	105
Table 4	106

1. Important safety precautions and warnings

⚠ Danger Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury. Indicates a potentially hazardous situation which, if not avoided, could result in serious injury and equipment damage.

This stimulator must **NOT** be used in combination with the following medical devices:

- Internally transplanted electronic medical devices, such as a pacemaker.
- Electronic life support equipment, such as respirators.
- Electronic medical devices attached to the body, such as electrocardiographs.
- This device **CANNOT** be used simultaneously with another TENS/EMS device. Using this stimulator with other electronic medical devices may cause erroneous operation of those devices.

⚠ Warning

Do not use this device under the following conditions

- Consult with your physician before using this device, because the device may cause lethal rhythm disturbances in certain susceptible individuals.
- On open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions (e.g., phlebitis, thrombophlebitis, varicose veins); or on top of, or in proximity to, cancerous lesions.
- Over areas of skin that lack normal sensation.
- On the opposite sides of your head since the effects of stimulation of the brain are unknown.
- If you are connected to high-frequency surgical equipment. This may result in burns on the skin under the electrode and may damage the device.
- Within less than 39 inches (1 meter) from shortwave or microwave medical equipment. Close proximity to this equipment may cause unstable device output.

Do not use on the following individuals

- Pregnant people, because the safety of electrical stimulation during pregnancy has not been established.
- Children or infants, because the device has not been evaluated for pediatric use.
- Persons incapable of expressing their thoughts or intentions.

Do not use this device during the following activities

- When in the bath or shower.
- While sleeping.
- While driving, operating machinery, or during any activity in which electrical stimulation can put you at risk for injury.

Pain management warnings

- If you have had medical or physical treatment for your pain, consult with your physician before using this device.
- If your pain does not improve, becomes seriously chronic or severe, contact your physician.
- The mere existence of pain functions as a very important warning telling us that something is wrong. Therefore, if you suffer from any serious illness, consult your physician in order to confirm that it is advisable for you to use this device.

Warnings and precautions regarding the electrodes

- Apply electrodes to normal, healthy, dry, clean skin (of adult patients) because it may otherwise disrupt the healing process.
- If you experience any skin irritation or redness after a session, **DO NOT** continue stimulation in that area of the skin.
- Ensure **NO** residual skin lotions or conductive gels are left on the skin prior to using the electrodes, as this can result in burns due to inadequate adhesion.
- If the electrodes are lifting or no longer stick to the treatment area, they **MUST** be replaced to avoid sudden shock or possible burns on any of the applied electrodes, including the electrodes adhered correctly and completely.

Never apply the electrodes to:

- The neck or any area of the throat because this could cause severe muscle spasms resulting in closure of the airway, difficulty in breathing, or adverse effects on heart rhythm or blood pressure.
- Both sides of the thorax simultaneously (lateral or front and back), or across your chest because the introduction of electrical current may cause rhythm disturbances which could be lethal.
- Stimulation should not be applied across or through the head, directly on the eyes, covering the mouth, or from electrodes placed on the chest and the upper back or crossing over the heart.
- Application of electrodes near the thorax may increase the risk of cardiac fibrillation.

Caution

- **DO NOT** bend or fold because the electrode may not function properly. Place the electrodes onto the plastic film and then store into the sealed package when not in use.
- **DO NOT** apply ointment or any solvent to the electrodes or to your skin because it will disrupt the electrodes from functioning properly. The electrodes are already pre-gelled and will adhere to your skin.
- To avoid damage to the adhesive surface of the electrodes, put the electrodes only on the skin or on the plastic film provided.
- Make sure the components are well connected and the electrodes are fixed on the part of the body you wish to treat or the therapy may not be effective.

Do not use the electrodes in the following way

- Electrodes should **NOT** touch each other when placed onto your skin.
- **DO NOT** place on your spine or backbone.
- Pad should **NOT** touch any metal object, such as a belt buckle or necklace.
- Electrodes should **NOT** be placed simultaneously on the soles of both feet.
- Electrodes should **NOT** be placed simultaneously on the calves of both legs.
- **DO NOT** share electrodes with another person. This may cause a skin irritation or infection. Electrodes are intended for use by one person.
- **DO NOT** place or relocate the electrodes while the device is on.
- **ALWAYS** turn the power off before removing or changing the pad location.
- **DO NOT** leave electrodes attached to the skin after treatment.

Caution while using the stimulator

- If the stimulator is not functioning properly or you feel discomfort, immediately stop using the device. If any type of shock or burn should occur, stop using immediately and consider seeking medical attention (if necessary).
- **DO NOT** use for any other purpose except for what it is intended for. Any other use is considered improper and may result in damage to property or even personal injury.
- **DO NOT** insert the electrode plug into any place other than the jack on the main device.
- **DO NOT** pull on the electrode cord during treatment.

- **DO NOT** use the device while wearing electronic devices such as watches as this may damage the device.
- **DO NOT** use near a cell phone as this may cause the stimulator to malfunction.
- **DO NOT** bend or pull the end of the cord.
- When pulling out the cord from the device, hold the plug and pull.
- Replace the cord when broken or damaged.
- **DO NOT** throw the batteries into a fire. The batteries may explode.
- Dispose of the device, batteries, and components according to applicable legal regulations. Unlawful disposal may cause environmental pollution.
- The size, shape and type of electrodes may affect the safety and effectiveness of electrical stimulation.
- The electrical performance characteristics of electrodes may affect the safety and effectiveness of electrical stimulation.
- Using electrodes that are too small or incorrectly applied, could result in discomfort or skin burns.
- **DO NOT** remove electrodes from treatment area until the device is turned off.

General Precautions

- The patient is an intended operator of the device.
- The patient can fully operate the device, change the battery and clean the device, but maintenance and modification of the device can only be done by a service technician authorized by the manufacturer.
- Please keep the battery compartment closed during operation and avoid touching the battery and the body part at the same time.
- The long-term effects of electrical stimulation are unknown.
- Apply stimulation to only normal, intact, clean, dry, and healthy skin.
- TENS is **NOT** effective in treating the original source or cause of the pain.
- TENS is **NOT** a substitute for pain medications and other pain management therapies.
- TENS devices **DO NOT** cure disease or injuries.
- TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism.
- Effectiveness is highly dependent upon patient selection by a practitioner qualified in the management of pain patients.
- You may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium (gel).
- If you have suspected or diagnosed heart disease, you should follow precautions recommended by your physician.

- If you have suspected or diagnosed epilepsy, you should follow precautions recommended by your physician.
- Use caution if you have a tendency to bleed internally, such as following an injury or fracture.
- If any type of shock or burn should occur, stop using immediately and seek medical attention (if necessary).
- Consult with your physician prior to using the device after a recent surgical procedure, because stimulation may disrupt the healing process.
- Keep device away from young children. The device contains small pieces that may be swallowed. The electrode cord can cause strangulation, immediately contact your physician should any of these things occur.
- Use this device only with the leads, electrodes, and accessories recommended by the manufacturer.
- **DO NOT** maintain or service the device while the device is in use.
- **DO NOT** modify this equipment without authorization of the manufacturer.
- The time required for the device to warm from the minimum storage temperature (0°C) between use until the device is ready for use at ambient temperature (20°C): about 2 hours.
- The time required for the device to cool from the maximum storage temperature (40°C) between use until the device is ready for use at ambient temperature (20 °C): about 2 hours.
- Check the device for signs of wear and tear or damage. If there are such signs of wear and tear or damage or if the device was used improperly, do not use it and contact manufacturer or retailer.
- Do not modify electrodes (e.g. by cutting them). Doing so increases the current density, which is potentially hazardous, an effective current density beyond 2 mA/cm² requires increased caution.
- Protect battery from excess heat; Take the battery out of the product if the product is not used for a long period of time.

Possible adverse reactions

- **DO NOT** use to treat one region for extended periods of time (more than 30 minutes a session, up to 2 times/day) or muscles in that region may become exhausted and sore.
- You may experience skin irritation and burns beneath the stimulation electrodes applied to your skin.
- You should stop using the device and consult with your physician if you experience adverse reactions from the device.

Applied part of type BF.

Disposal in accordance with Directive 2012/19/EU (WEEE).

This product satisfies the applicable European and national requirements.

The name and the address of the manufacturer.

Manufacturer:

Name: Shenzhen Dongdixin Technology Co., Ltd.

Add: Floor 1-2, No. 3 Building, Fanshen Xusheng
Industrial Estate, Xilixiaobaimang, Nanshan District,
518108, Shenzhen, PEOPLE'S REPUBLIC OF CHINA
e-mail service@e-caretalk.com

EU

REP

The name and the address of the authorized EU-representative.

Authorized EU-representative:

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537 Hamburg Germany
Tel: 0049-40-2513175
Fax: 0049-40-255726

LOT

Batch code

SN

Serial number

Refer to Instruction Manual.

IP22

The first 2: Protected against solid objects greater than 12.5 mm in diameter.

The second 2: Protected against drops of water falling when the product is inclined up to 15 degrees from the vertical. Vertically falling drops shall have no harmful effects when the enclosure is tilted at any angle up to 15° on either side of the vertical.

Medical device

Unique device identifier

Keep dry

Keep away from sunlight

General symbol for recovery/recyclable

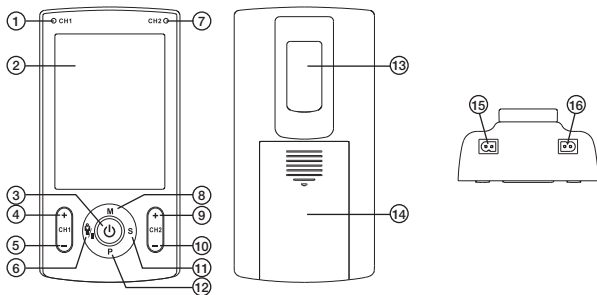
Date of manufacture

2. The package contains

- TENS/EMS machine
- 2 electrode cables, each with 2 connections for electrode pads
- 4 electrode pads, 40 × 40 mm
- User Guide and Quick Guide
- 4 × AAA/LR03 batteries


3. Product description

1. CH1 – LED, lights when channel 1 is enabled
2. LCD display
3. [Power] Switch
 - Press to switch on, the display lights up.
 - Press and hold for about 3 seconds to switch off, the display goes out.
 - In treatment mode: Stop the treatment.
 - In memory mode: Exit memory mode.
 - In setting mode: Exit from the setting state.
4. [CH 1+] Increase the intensity on channel 1. Adjustment from 0 (off) to max.
5. [CH 1-] Decrease the intensity on channel 1. Adjustment from max to 0 (off). In setting mode: Select the next parameter.

6.  Select user, check data in memory or pause the treatment.
 - Select user U1 (user 1) or U2 (user 2).
 - In memory mode: Check data from previous treatments stored in memory.
 - In treatment mode: Pause and resume the treatment.
7. CH2 – LED, lights when channel 2 is enabled
8. [M] In standby mode, select the treatment: TENS – MASS – EMS.
9. [CH 2+] Increase the intensity on channel 2. Adjustment from 0 (off) to max. In setting mode: Adjust the user program parameter.
10. [CH 2-] Decrease the intensity on channel 2. Adjustment from max to 0 (off). In setting mode: Adjust the user program parameter.
11. [S] Confirm the selected users, select the treatment area, open or delete data in memory.
 - Confirm the selected user U1 or U2.
 - Choose the treatment area. Press to step between 9 different treatment areas.
 - Press and hold for 3 seconds to open memory mode (in standby mode).
 - Press and hold for 3 seconds and confirm deletion of memory data (in memory mode).
12. [P] Standby mode: Select the treatment programme before the treatment starts.
 - The programmes have a different wavelength, pulse rate and pulse width. Diagrams with a description of all programmes can be found at the end of the User Guide.
13. Belt clip
14. Battery cover
15. Connection for channel 1
16. Connection for channel 2

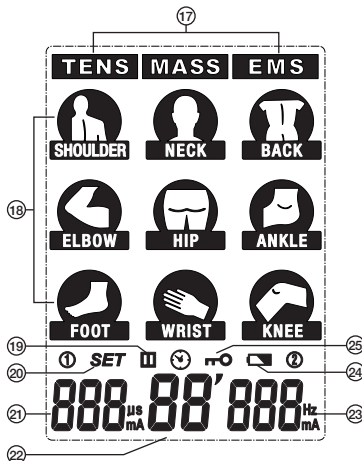4. Display

- 17. TENS, MASS or EMS shows the chosen treatment.
- 18. Treatment area on the body: Shoulder, neck, back, elbow, hip, ankle, foot, wrist and knee. (Does NOT show where the electrode pads should be placed on the body).
- 19. Pause indicator
- 20. Settings mode symbol
- 21. Pulse duration and intensity for channel 1.
- 22. Treatment duration
- 23. Pulse frequency and intensity for channel 2.
- 24. Battery charge status indicator.
- 25. The intensity level keypad lock is activated.

Note: If the electrode pads are not attached to the skin, the intensity can only be adjusted up to 9, at 10 the machine returns to 0 again. When the electrode pads are attached to the skin, the intensity can be adjusted between 0 and maximum value.

5. Functions

5.1 Treatments

Intended use:

- The device is designed to be used for temporary relief of pain, including the acute and chronic pain relief.
- The device is designed to be used for stimulate muscles for prevention of muscular atrophy, increase of blood flow circulation, muscle strengthening and improve and facilitate muscle performance.

Electrical muscle stimulation is used by more and more sportsmen and sportswomen to improve their performance. This is achieved by increasing muscle strength, volume and endurance.

- TENS (Transcutaneous Electrical Nerve Stimulation): Electro therapy with low-intensity and short impulses to e.g. relieve pain.
- EMS (Electrical Muscle Stimulation): Electrical muscle stimulation which increases blood circulation.
- MASSAGE Electrical stimulation that stimulates muscles in the treatment area to mild activity which in turn results in an improved sense of well-being.

5.2 Treatment areas

Shoulder, neck, back, elbow, hip, ankle, foot, wrist and knee.

5.3 Treatment programmes

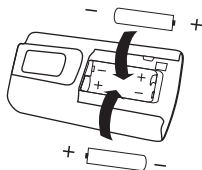
- TENS has 3 programmes: P1, P2 and U1 for each treatment area.
- EMS has 3 programmes: P1, P2 and U1 for each treatment area.
- MASSAGE has 4 programmes: P1, P2, P3 and P4 for each treatment area.

The programmes have a different wavelength, pulse rate and pulse width.

6. Getting started

6.1 Insert the batteries


1. Open the battery cover by pressing it in while sliding in the direction of the arrow.
2. Insert 4 × AAA/LR03 batteries in the battery compartment as shown by the marking.

6.2 Connect the cables to the electrode pads


1. Insert the cable connectors into the electrode pad connectors.
2. Push the connectors together so that no metal is visible on any connection.

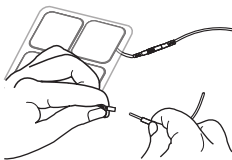Warning: Only use electrode pads that are marked with CE.

6.3 Connect the cables to the TENS/EMS machine

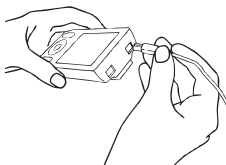1. First make sure the machine is switched off.
2. Grasp an insulated part of the connector and connect this to one of the outlets (15) or (16) on the top.
3. It is possible to use one channel with two electrode pads, or both channels and thus with four electrode pads at the same time. If you use both channels, two different areas of the body can be stimulated simultaneously.

6.4 Place the electrode pads on the skin

The electrode pads should only be used by one person for reasons of hygiene. Additional electrode pads are available as an accessory (our art. no 34-4103-1).

Make sure that the skin is completely dry and clean.

1. Remove the protective backing plastic from the self-adhesive surface. Save the protective backing plastic and refit to the electrode pads after use.
2. Wash the skin thoroughly with soap and water, wipe dry and then attach the self-adhesive electrode pads. Make sure they make full contact over the entire surface.
3. Start the TENS/EMS machine when the electrode pads have been attached to the skin.

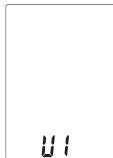
Warning: Never remove the electrode pads from the skin when the TENS/EMS machine is switched on.

Note: See section 11. *Placement of the electrode pads* for information about the exact placement of the electrode pads on the body.

6.5 Switch on the TENS/EMS machine

Read the safety instructions before switching on the machine. Pay particular attention to circumstances when the machine must not be used. The TENS/EMS machine is not a toy, but a powerful treatment aid.

Switch on the machine: Press , the display lights up.

6.6 Select the user

There is a memory for 2 different users (U1) and (U2), so that 2 people can use the product with individual settings.

1. Press  to select user 1 or 2.
2. Press [S] to confirm the selected user.

6.7 Choose a treatment and treatment area

1. There are 3 different treatments: **TENS**, **MESSAGE** or **EMS**. Press [M] to select the required treatment.
2. There are 9 different treatment areas. Press [S] to select the treatment area.
3. The selected treatment and the treatment area flash.

6.8 Select the treatment programme

For the 3 treatments and the 9 treatment areas there is a total of 78 treatment programmes. TENS and EMS has 27 programmes and MESSAGE has 24 programmes.

Press [P] and select the treatment programme (**P1**, **P2**, **P3**, **U1*** or **P4**) for the treatment area you have chosen. **P1**, **P2**, **P3**, **U1** or **P4** is shown at the bottom of the display for the selected programme.

The treatment programmes have a different wavelength, pulse rate and pulse width. Diagrams with a description of all programmes can be found at the end of the User Guide.

* See section 6.16 *Individual settings U1*

6.9 Start the treatment and adjust the intensity

1. Press [CH1 +] or [CH 2 +] to increase the output intensity from **0** and upward for channel 1 or channel 2 and in doing so, start the treatment.
2. Press [CH1 -] or [CH2 -] repeatedly to reduce the output intensity for channel 1 or channel 2. Adjustment occurs in 1 mA increments.
3. The set intensity (mA) and the remaining treatment time are shown at the bottom of the display.

The maximum treatment duration is 90 minutes.

Warning: Reduce the stimulation level if you experience discomfort.

Seek medical attention if the discomfort persists.

6.10 Pause the treatment

Press [P/■] to pause the treatment. Press again to continue.

6.11 Switch off the machine

1. Press and hold [O] for about 3 seconds to switch off. **Note:** Except for working mode, the device will automatically turn off when no button is pressed in 3 minutes.
2. Carefully remove the electrode pads from the body and fit the protective backing plastic. If the treatment is to be continued within a short time, you may leave the electrode pads on the body.
3. Wash the skin with mild soap and water when the electrode pads have been removed.

Store the electrode pads in a plastic bag.

Seek medical attention if the skin is irritated.

6.12 Quick stop

If you need to switch off the machine quickly:

Press [P/■] to pause the treatment, press again if you wish to continue.

Note:

- The strength of stimulation must be adjusted individually for each user.
- The intensity can be adjusted from **0** (off) to max.
- Switch off the machine before the electrode pads are removed from the body.
- The intensity goes down to 0 automatically when the treatment is finished.

6.13 Keypad lock

Note: If you do not press any buttons for 20 seconds this will enable a security lock/keypad lock that locks the buttons so that the intensity cannot be increased and no button functions can be used without disabling the keypad lock. A key symbol is shown on the display when the keypad lock is enabled. To turn off the keypad lock: Press [CH1-] or [CH2-].

6.14 Check what is stored in memory

The machine saves different parameters in memory such as the memory number of the treatment and its programme, treated body part, average intensity and duration. A maximum of 30 treatments may be stored in memory, after 30 treatments, the oldest is deleted and the most recent saved.

To check what is stored in memory: Press and hold [S] for 3 seconds to open memory mode. Press [M/■] to read the various memory parameters. Press [CH1] to go to the next memory number.

Note:

- The machine returns to standby mode when you press [P], or automatically after 30 seconds.
- Press and hold [S] for 3 seconds to show the total number of treatments in the memory, they are shown flashing on the display:
 - Delete everything in memory: press [M/■].
 - Cancel deletion and return to standby mode: press [P].

6.15 Battery warning

When the battery symbol flashes, the batteries should be replaced as soon as possible. The machine will continue to work for a few hours before shutting down.

6.16 Individual settings U1

In TENS and EMS modes there is a mode (U1) for programming your own individual treatment settings.

1. Turn the machine on and select U1 by pressing [P].
2. Keep [P] held in to enter the settings mode for U1.
3. Select the desired parameters using [CH1-].
4. Adjust the selected parameter using [CH2+] or [CH2-].
 - Adjust the pulse duration from 50 to 350 μ s.
 - Adjust the pulse frequency from 1 to 150 Hz.
 - Adjust the treatment duration from 5 to 90 minutes.
5. Press [P] to confirm settings and enter standby mode.

Note: Reset U1 to default factory settings by holding in [CH2-] at the same time as you insert the batteries.

7. Care and Maintenance

Remove the batteries before cleaning. Wipe off the machine with a soft, damp cloth. Use a mild detergent if necessary, never a strong detergent or solvent.

Note:

- Check the electrode pads before each use, and replace if damaged in any way.
- Store the electrode pads in a plastic bag between treatment sessions.
- Do not expose the device to direct sunlight, high temperature, heat radiated from a heat radiator, excessive amounts of dust, moisture, vibrations and mechanical shocks.

8. Troubleshooting

Problem	Probable cause	Solution
The display does not light up.	The batteries are spent or inserted incorrectly.	1. Replace the batteries. 2. Check whether the batteries have been installed correctly.
Weak stimulation.	1. The electrode pads have dried or are dirty. 2. Wrong placement. 3. Faulty cables.	1. Replace with new electrode pads. 2. Check the electrode pad placement. 3. Check the cables and ensure they are connected properly.
Stimulation is uncomfortable.	1. Intensity too high. 2. The electrode pads sit too close together. 3. Damaged electrical pads or cables. 4. The electrode pads' active area is too small.	1. Reduce the intensity. 2. Change the placement. 3. Replace them. 4. Replace the electrode pads if smaller than 4 × 4 cm.

Intermittent output.	1. Faulty cables or connection. 2. Program function.	1. Check the cable connections, push them in fully. Reduce the intensity. Disconnect the cable and reconnect. Replace cables that are faulty. 2. Some programmes may be perceived as intermittent, read the programme description.
Stimulation is ineffective.	Placement of the electrode pads incorrect, check.	Read the description and change the placement of the electrode pads.

9. Responsible disposal

This symbol indicates that this product should not be disposed of with general household waste. This applies throughout the entire EU. In order to prevent any harm to the environment or health hazards caused by incorrect waste disposal, the product must be handed in for recycling so that the material can be disposed of in a responsible manner. When recycling your product, take it to your local collection facility or contact the place of purchase. They will ensure that the product is disposed of in an environmentally sound manner.

Batteries and accumulators that are not firmly enclosed in the electrical or electronic appliance and can be removed without destroying them must be separated from the appliance before it is handed in at a collection point and disposed of in a designated disposal facility.

Please dispose of the device in accordance with the legal obligation. If you have any questions, Please contact the local authorities responsible for waste disposal.

10. Specifications

Material	ABS plastic
Channels	2 channels
Power supply	4 × AAA/LR03 batteries
Waveform	Two phases square wave
Pulse length	30–350 µs
Pulse frequency	1–290 Hz
Treatment duration	5–90 minutes
Intensity	Adjustable from 0 to 90 mA (at 1000 ohms)
Operating environment	5 to 40 °C at 30–75% RH and air pressure 700–1060 hPa
Storage and transport condition	0 to 40 °C at 10–90% RH and air pressure 700–1060 hPa
Dimensions	approx. 117 × 60 × 25 mm (without belt clip)
Weight	approx. 96 g (without batteries)

Service life of the device: 3 years.

Service life of the battery: With new super heavy duty batteries, they will last for approx. 20 days when used 30 minutes per day in TENS NECK P01 Program at 40 mA intensity.

Service life of the electrode pads: pads can be cleaned and reused for up to 10–15 times.

Applied part: Electrode

IP classification rating: IP22

Software Version: V1

Classification of ME Equipment: Internally powered/Continuous

Warning: Read the entire User Guide before use.

Unit type BF.

The product must be handled in accordance with decree 2012/19/EU – WEEE (Waste Electrical and Electronic Equipment).





CE 0197





The product is in conformity with the guidelines set down in the European Medical Device Directive (93/42/EEC) and as amended by Directive 2007/47/EC.






Notified Body TÜV Rheinland (CE0197).






The product conforms to the requirements set out in the following European standards: EN 60601-1, EN 60601-1-2, EN 60601-2-10, ISO10993-5, ISO10993-10

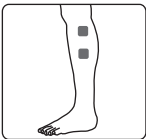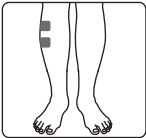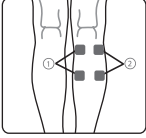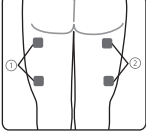11. Placement of the electrode pads

Electrode placement	Treatment position
	In upright sitting position with feet resting on the floor.
	Upright sitting position with feet resting on the floor.
	Upright sitting position with feet under a piece of furniture to avoid bending of ankles.
	Upright sitting position with support for feet and back (e.g., sitting in a door frame).
	Lying flat on one's stomach with ankles comfortably immobilised.

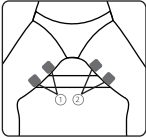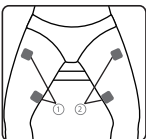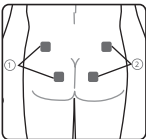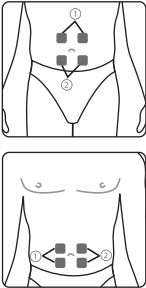	Upright sitting position, knees pressed together around a hard (but comfortable) object.
	Upright sitting position, knees pressed together around a hard (but comfortable) object.
	Lying on one's stomach or standing up.
	Lying on one's stomach or standing up.

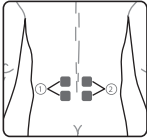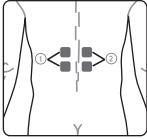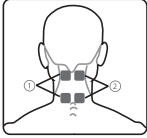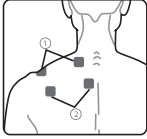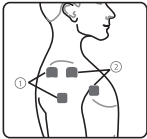	Upright sitting position.
	Upright sitting position.
	Upright sitting position.
	Upright sitting position.
	Upright sitting position with elbows pressed against side of the body (inside chair armrests).

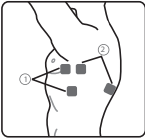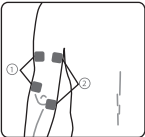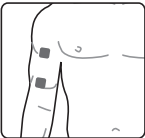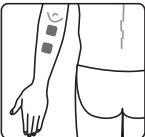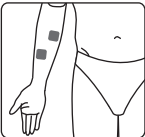	Upright sitting position with elbows pressed away from the body (outside chair armrests).
	Upright sitting position with arms resting on chair armrests.
	Upright sitting position with lower arms resting on chair armrest and palms of hands upward.
	Upright sitting position with lower arms resting on chair armrests and palms resting on chair armrests.
	Upright sitting position with lower arms resting on chair armrests and palms of hands upward. Hands clasping some object so fingers are slightly bent.

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Appendix/Bilaga/Liite 1 TENS

Program Therapy part	P1			P2			U1 (Default)			Treatment time (min)
	Pulse rate (Hz)	Pulse width (µs)	Waveform	Pulse rate (Hz)	Pulse width (µs)	Waveform	Pulse rate (Hz)	Pulse width (µs)	Waveform	
SHOULDER	2~80	100~200	Simple modulated pulse	50	200	Synchronous	2~125	100~200	Simple modulated pulse	30
NECK	2	180	Continuous	80	70~180	Pulse width modulated	80	180	Continuous	
BACK	80/2	180	Hans	80	70/180	Hans	100	330/200	Amplitude modulated	
ELBOW	2	180	Continuous	80	70~180	Pulse width modulated	2~125	100~200	Simple modulated pulse	
HIP	100	330/200	Amplitude modulated	125	330/200	Amplitude modulated	80	330/200	Amplitude modulated	
ANKLE	2~60	100~200	Simple modulated pulse	2~8	300	Pulse rate modulated	2~40	100~200	Simple modulated pulse	
FOOT	80	70~180	Pulse width modulated	80	70/180	Hans	2~100	100~200	Simple modulated pulse	
WRIST	50	200	Synchronous	65	200	Synchronous	2~80	100~200	Simple modulated pulse	
KNEE	50	350	Asynchronous	2	180	Continuous	80	200	Alternate Ramped Burst	

Appendix/Bilaga/Lite 2 MASSAGE

Therapy part	program
SHOULDER	P1, P2, P3, P4
NECK	P2, P3, P4
BACK	P1, P2, P3
ELBOW	P2, P3
HIP	P1, P2, P3, P4
ANKLE	P3, P4
FOOT	P3, P4
WRIST	P2, P3
KNEE	P1, P2

Program	Phase	Pulse rate (Hz)	Pulse width (µs)	Work time (s)	Rest time (s)	Treatment Time(min)
P1	1	8	100	4	/	30
P2	1	25	30~220	3.5	1.0	
	2	25	30~220	2.5	0.9	
	3	33	30~220	1.9	0.9	
	4	43	30~220	1.3	0.8	
	5	53	200	0.9	0.7	
	6	69	200	0.7	0.6	
	7	79	200	0.5	0.5	
	8	69	200	0.7	0.6	
	9	53	200	0.7	0.6	
	10	43	30~220	1.3	0.8	
	11	33	30~220	1.9	0.9	
	12	25	30~220	2.5	0.9	
	13	25	30~220	3.5	1	
P3	1	83	50~220	4	1	
	2	100	50~220	3.8	0.8	
	3	111	50~220	3.1	0.7	
	4	118	50~220	2.6	0.6	
	5	132	50~220	2.3	0.6	

Program	Phase	Pulse rate (Hz)	Pulse width (μ s)	Work time (s)	Rest time (s)	Treatment Time(min)
P3	6	118	50~220	2.6	0.6	30
	7	111	50~220	2.8	0.7	
	8	100	50~220	3.3	0.8	
P4	1	147	30~220~150	12.0	1.0	
	2	169	30~220~150	10.3	0.9	
	3	196	30~220~150	8.5	0.6	
	4	237	30~220~150	6.8	0.6	
	5	285	30~220~150	5.1	0.4	
	6	290	30~220~150	5.7	0.5	
	7	238	30~220~150	6.3	0.5	
	8	197	30~220~150	8	0.6	
	9	191	30~220~150	8.5	0.7	
	10	168	30~220~150	9.1	0.8	
	11	150	30~220~150	10.8	0.9	

Appendix/Bilaga/Liite 3 EMS

Program Therapy part	P1			P2			U1 (Default)			Treatment time (min)
	Pulse rate (Hz)	Pulse width (μs)	Waveform	Pulse rate (Hz)	Pulse width (μs)	Waveform	Pulse rate (Hz)	Pulse width (μs)	Waveform	
SHOULDER	80	150	Synchronous	50	200	Synchronous	40	200	Synchronous	30
NECK	1	50	Continuous	2~60	100~200	Simple modulated pulse	2~8	300	Pulse rate modulated	
BACK	50	350	Synchronous	65	350	Synchronous	80	150	Synchronous	
ELBOW	50	150	Synchronous	50	200	Synchronous	50	350	Asynchronous	
HIP	65	350	Synchronous	40	200	Synchronous	50	350	Synchronous	
ANKLE	50	200	Synchronous	65	200	Synchronous	50	200	Synchronous	
FOOT	50	200	Synchronous	50	200	Synchronous	65	200	Synchronous	
WRIST	50	200	Asynchronous	50/8	200	Synchronous	50	350	Asynchronous	
KNEE	50	350	Synchronous	50	350	Synchronous	40	350	Synchronous	

Appendix/Bilaga/Liite 4 EMC

Electromagnetic Compatibility (EMC)

- This device should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, this device should be observed to verify normal operation in the configuration in which it will be used.
- Use of accessories other than those specified or provided by the manufacturer of this device could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- When the operating environment is relatively dry, strong electromagnetic interference usually occurs. At this time, the device may be affected as follows:
 - The device stops output.
 - The device turns off.
 - The device restarts.

The above phenomenon does not affect the basic safety and essential performance of the device and the user can use it according to the instruction. To avoid the above phenomenon, please use the device in accordance with in the manual specified environment.

Table 1

Declaration - electromagnetic emission		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

Table 2

Declaration - electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV line(s) to lines ± 0.5 kV, ± 1 kV, ± 2 kV line(s) to earth	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0° 0 % UT; 250/300 cycles	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: UT is the a.c. mains voltage prior to application of the test level.			

Table 3

Declaration - electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 V 0.15 MHz to 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz	Not applicable	Portable and mobile RF communications equipment should be used no closer to any part of device, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2\sqrt{P}$ 150 kHz to 80 MHz $d = 1.2\sqrt{P}$ 80 MHz to 800 MHz $d = 2.3\sqrt{P}$ 800 MHz to 2.7 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz	10 V/m	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should ^a be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed RF transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which device is used exceeds the applicable RF compliance level above, device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating device. Over the frequency range 0.15 MHz to 80 MHz, field strengths should be less than 3 V/m.			

Table 4

Recommended separation distances between portable and mobile RF communications equipment and device			
The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and device, as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter, W	Separation distance according to frequency of transmitter, m		
	0.15 MHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.7 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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